

**BAUSCH+LOMB**



THERA°PEARL combines the healing power of **HOT** & **COLD** therapy into one convenient pack. Enjoy soothing relief from your everyday aches & pains and feel the optimal therapeutic benefit as our Pearl Technology® conforms to your body.

**INTENDED USE:**

Reusable therapeutic hot & cold pack for use with common aches, pains, swelling, and bruising.

**COLD THERAPY**

To help relieve puffy eyes, dark circles/bags under eyes, swelling, headaches, migraines, and sinuses. Freeze for at least 2 hours.

**HEAT THERAPY**

To help relieve dry eyes and styes. Heat in microwave for specified time.

Suggested heating time for (1) one THERA°PEARL Eye Mask

700 Watts ~ 15 Seconds
1000 Watts ~ 12 Seconds
1250 Watts ~ 10 Seconds

Clean eye mask gently with mild soap after each use.

See full directions on eye mask prior to use and application.

**CAUTION:** For external use only. Do not use on infants. Do not ingest contents. Do not use for purposes other than intended. Check with your doctor before use on broken, irritated, highly sensitive or insensitive skin. Use only under adult supervision. Always test pack temperature prior to application. Overheating or prolonged application may cause injury to person, including burns. Heating beyond suggested time may damage or destroy TheraPearl pack.